

LimbLogic® Clinician App

An iOS app is now available for the prosthetist to control LimbLogic.

Note: once the Android version of the app is available, the fob that has previously been used for controlling LimbLogic will become an optional accessory and will only be included with LimbLogic when specifically ordered.


INSTALL THE APP

1. Open the App Store on your iOS device.
2. Search for the **LimbLogic CP** app shown at right.
3. Tap **GET** to download.



GET STARTED

1. Turn the pump on. Wait 5 seconds. Open the **LimbLogic** app.
2. The initial screen will display the number of pumps in range. Tap **+** to pair a pump in range with the app.
3. Select **Left** or **Right** to indicate which leg the pump is on. Enter the **Patient ID** and pump **Serial Number**. Tap **Save**.

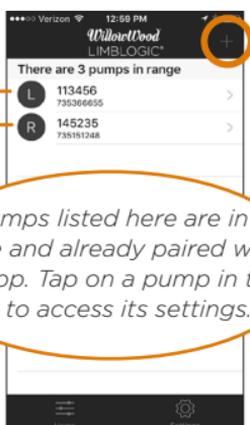

Pumps listed here are in range and already paired with the app. Tap on a pump in the list to access its settings.



The pump will now appear in the list of pumps in range.

USE THE APP

MODE CONTROL

Switch between **Active** and **Standby**.

Active: Pump turns on as needed to maintain the vacuum level.

Standby: Pump turns off until switched back to **Active**.

BLUETOOTH® STATUS


Blue = connect to pump.
Grey = not connected.

EDIT the pump's Patient ID, Serial Number, or side.

VACUUM PUMP BATTERY STATUS

If this symbol is red, charge the pump battery soon.

CURRENT VACUUM LEVEL

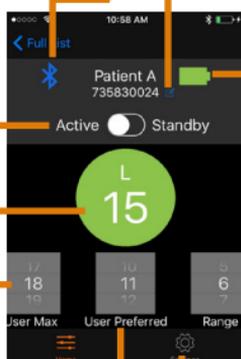

CHANGE VACUUM RANGE

Scroll up or down to change how much the current vacuum level can fall below the **User Preferred Setting** before the pump activates.

CHANGE USER MAXIMUM

Scroll up or down to change the limit you want to place on the **User Preferred Setting**.

SETTINGS (page 2)

CHANGE USER PREFERRED SETTING

Scroll up or down to change the vacuum level the patient would like to maintain in the socket. Also known as **Vacuum Set Point**.

SETTINGS

Tap the **Settings** icon  to display the following options:



Tap the **HELP** icon to view the instruction manual, e-mail a question to WillowWood, or visit the LimbLogic website. You may also choose to **Display Help at Startup** if desired.



Tap the **ABOUT** icon to view information about the revision level of the app.



Tap the **PRIVACY** icon to view the Terms & Conditions for the app.

Tap the  icon to exit the **Settings** screen.



The Clinician's app is intended for certified clinicians and should not be used by untrained users. If you are a patient with a LimbLogic device and wish to link to your pump with a smart device, please download the Patient app.



This insert is intended for informational purposes. For complete information about the use of your LimbLogic products, please refer to the Prosthetist and Patient Manuals or contact WillowWood.

WillowWood[®]

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands
Patent [www.willowwoodco.com/
education-and-resources/patents](http://www.willowwoodco.com/education-and-resources/patents)



PN-2487-A 10 OCT 2016